11001001

EN USER MANUAL

Braster System – in-home breast examination system

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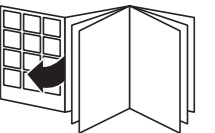
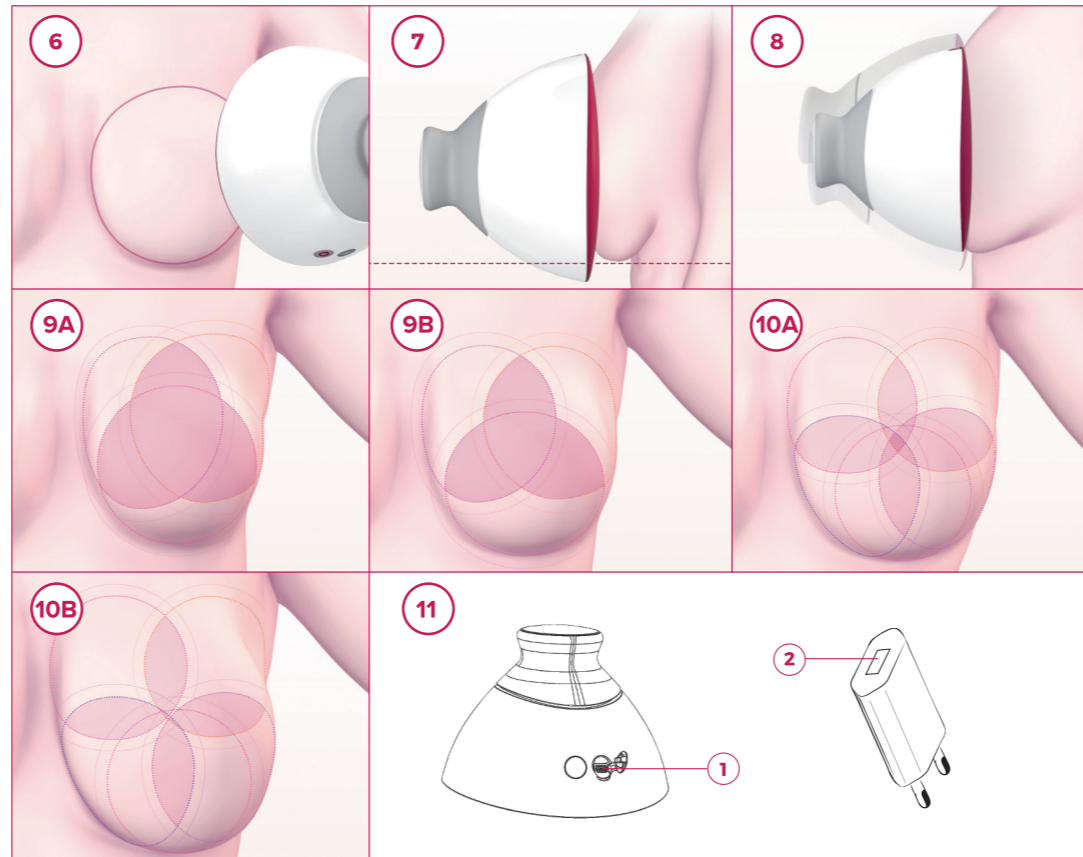
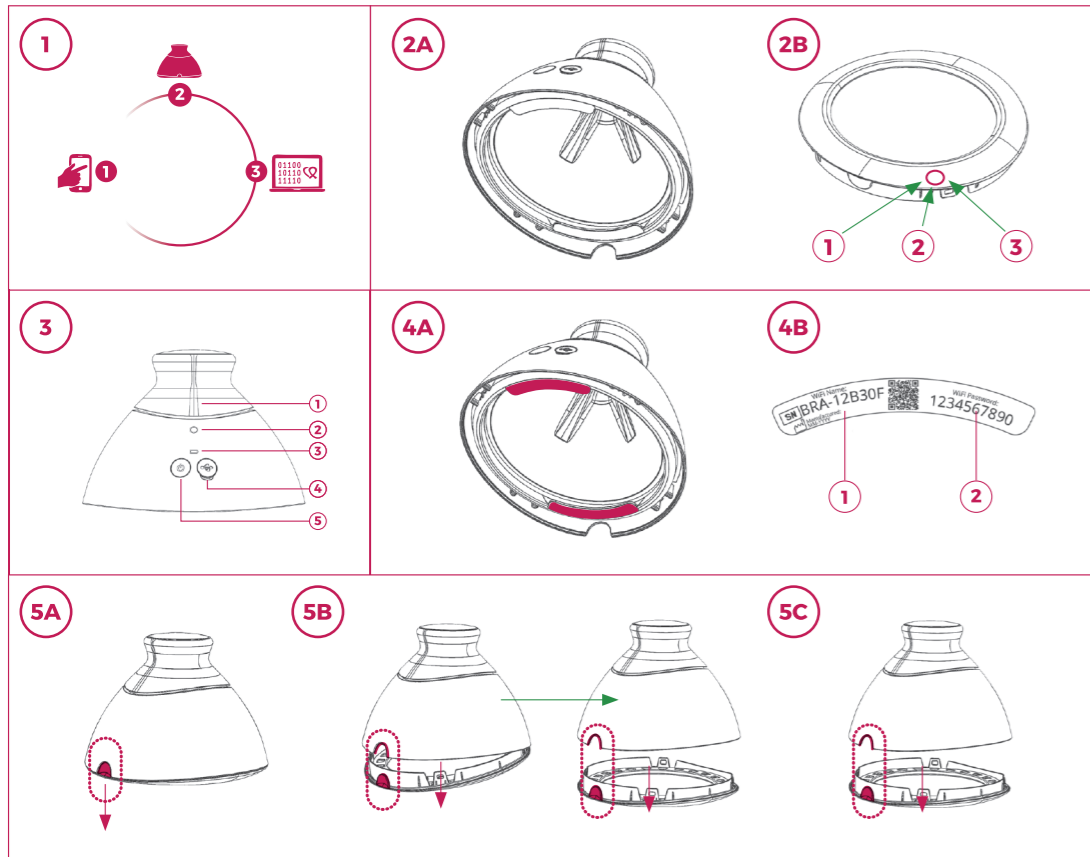
EXPLANATION OF SYMBOLS USED IN THE USER MANUAL, ON THE LABEL AND ON THE PACKAGING

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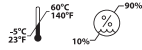
USER MANUAL

TABLE OF CONTENTS

GLOSSARY	8
IMPORTANT SAFETY INFORMATION	9
INTRODUCTION	10
1. DESCRIPTION AND INTENDED USE	11
1.1. DESCRIPTION	11
1.2. INTENDED USE	11
1.2.1. INDICATIONS	11
1.2.2. CONTRAINDICATIONS	12
2. OPERATING PRINCIPLES OF THE DEVICE	12
3. PACKAGE CONTENTS	13
4. OVERVIEW OF THE BRASTER DEVICE INTERFACE	13
5. MATRIX REPLACEMENT	15
6. DEVICE PREPARATION PRIOR TO EXAMINATION	15
6.1. PREPARATION OF THE BRASTER DEVICE BEFORE THE FIRST EXAMINATION	15
6.2. PREPARATION OF THE BRASTER DEVICE BEFORE SUBSEQUENT EXAMINATIONS	16
7. EXAMINATION	16
7.1. IMPORTANT INFORMATION	16
7.2. HOW TO PERFORM AN EXAMINATION	17
7.2.1. PREPARATION	17
7.2.2. QUESTIONS	18
7.2.3. TURNING THE DEVICE ON/OFF	18
7.2.4. DEVICE CONNECTION	19
7.2.4.1. CONNECTING TO AN iOS DEVICE	19
7.2.4.2. CONNECTING TO AN ANDROID DEVICE	19
7.2.5. ACCLIMATIZATION	20
7.2.6. MATRIX SELECTION	20
7.2.7. EXAMINATION	21
7.2.8. SENDING THE RESULTS TO THE TELEMEDICAL CENTER	23
8. TROUBLESHOOTING THE MOBILE APP	23
9. CHARGING	24
10. SAFE OPERATION	25
11. DISINFECTION	25
12. CLEANING AND MAINTENANCE	26
13. SERVICING AND TECHNICAL SUPPORT	26
14. DECLARATION OF CONFORMITY	27
15. TECHNICAL AND OPERATIONAL SPECIFICATION OF THE DEVICE	27
16. ADVERSE EVENT AND MEDICAL INCIDENTS	28
17. FCC REGULATORY STATEMENT	29
18. CANADIAN REGULATORY STATEMENT	29
19. DISPOSAL	30

GLOSSARY

Braster automatic interpretation (Braster AI) – software responsible for analyzing the thermal images, along with the medical information provided by the user via the Braster Care mobile application.

Braster device or device – a device composed of an image acquisition device and a thermographic matrix manufactured by Braster S.A.

Braster e-store – www.braster.eu where the Braster device, examination packages and the set of thermographic matrices can be purchased. Please note that in some markets the Braster device as well as examination packages and other services can be purchased only from a local distributor.

Examination package – a pre-paid package of automatic thermogram interpretations, encompassing a specific number of interpretations.

Image acquisition device – the device used in the examination. On the image acquisition device, there is an ON/OFF button, a USB slot and an grip. The device is also fitted with electronics and image acquisition system. The inner part of the device consists of a blackened chamber which ensures the appropriate quality of the thermographic images. The image acquisition device is an electronic device and is controlled by a mobile app.

Liquid crystal foil – an element of the liquid crystal matrix. The foil contains liquid crystals, which map thermal variations in the breast, creating color images (thermograms).

Liquid crystal thermographic matrix (or liquid crystal matrix or thermographic matrix) – an element of the Braster device composed of a liquid crystal foil and a plastic clamp. The thermographic image is generated on the inner side of the foil surface, i.e. on the side facing inwards (towards the inside of the device). The matrix is an applied part of the device, which means that it comes into contact with the skin of the breast.

Mammography – a radiological examination of the breast, performed on women, in which a series of pictures of the mammary gland is taken using x-rays. With its ability to visualize a number of characteristic changes and lesions, mammography is used for detection of breast cancer and other nipple pathologies before they manifest clinically. The efficacy of the examination is strictly connected with the structure of the breast, for that reason it is recommended for women older than 50 years of age.

Mobile app – the Braster Care app, software which is needed to perform the breast examination with the Braster device. It needs to be downloaded onto the user's mobile device (smartphone or tablet) before the first examination. The app is free of charge and available via Google Play (for Android) or via the App Store (for iOS).

Mobile device – a smartphone or tablet (for system requirements, go to: <https://www.braster.eu/system>).

My Account – an account on myaccount.braster.eu, where the user must register in order to conduct the examinations. In My Account the user can, for example, check the results of a thermographic examination.

Telemedical center – BRASTER'S S.A. infrastructure in which the users' data is stored and where the software needed for the provision of the services, in particular the results of examinations, offered by BRASTER S.A. is kept.

Thermogram (or thermographic image) – a graphic representation of the thermal image created on the surface of the liquid crystal foil which maps temperature distribution across the breast.

Ultrasound – a non-invasive method in which ultrasound is used to visualize tissue. Ultrasound imaging is a basic diagnostic tool used for detecting pathological changes in breast, including breast cancer.

User – a woman who uses the Braster System – in-home breast examination system.

IMPORTANT SAFETY INFORMATION

This section is provided to familiarize the user with critical information needed before the device is used. Additional warnings and precautions are also given in the other sections of the User Manual.

WARNINGS

Warnings are statements that alert the user to the possibility of injury, death, or other serious adverse reactions associated with the use or misuse of the device.

! It is important to note that an examination with the use of Braster System does not replace existing diagnostic methods currently used in clinical practice (such as mammography, ultrasound, MRI). Be sure to undergo routine mammography or ultrasound screening as recommended by your physician.

! Any use, operation and servicing of the device breaching this User Manual is prohibited and may cause damage to the device or disrupt its operation. The device must be used and operated as stipulated in the User Manual and serviced in a place indicated by BRASTER S.A. The manufacturer will not be held responsible for any consequences of improper use of the device.

! The device must not be charged while an examination is in progress. Use of the device while charging may lead to electrocution. Avoid contact with the device while it is being charged.

! Do not use the device if the housing of the device is damaged, as that may cause electrocution.

! The device should be charged using a USB Power Adapter (parameters: DC 5V 1A) which complies with IEC 60950-1 or IEC 60601-1. Switch the device off before charging it.

CAUTIONS

Cautions are statements that alert the user to the possibility of a problem with the device associated with its use or misuse, i.e. device failure, device malfunction, damage to the device or data loss.

! To ensure the safety and appropriate operation of the device, read the information on precautionary measures set out in the chapter of this User Manual titled "Important Safety Information" and other information included herein before using the device. The illustrations and screenshots used in this User Manual may slightly differ from the actual appearance of the device and mobile app.

! According to BRASTER's most recent studies thermographic matrices lose their properties after 36 months from their first use. If you use the matrices after the lapse of 36 months from their first use, a reliable interpretation of the examination will not be possible. To ensure proper operation of the device, replace the set of matrices after 36 months since the first use. However, please note that BRASTER S.A. still conducts studies on the matrices lifetime and the lifespan can be extended even further in the future.

! The device and matrices may be damaged if stored in a hot and/or moist place and/or in direct sunlight. Store them in a dry, cool and shaded place.

! You will pay for data transfer in accordance with your operator's tariff.

! You must only use the parts indicated in the User Manual and connect the Braster device to devices that are mentioned in the User Manual. Do not modify the Braster device. If you use parts other than those indicated in the User Manual or if you modify the Braster device or connect it to devices that are not mentioned in the User Manual, you may encounter problems with its operation and/or cause damage that will be attributed to the user, for which the manufacturer will not be held responsible.

! Charge the device for at least 30 minutes before the first examination.

INTRODUCTION

This User Manual comes with the Braster System – in-home breast examination system intended for breast self-examination by the user. Read the User Manual before using the device for the first time. The User Manual contains the necessary information on all of the functions and features of the device, its safety and the breast examination process.

It is recommended that you retain this User Manual for future reference.

BRASTER S.A. is the owner of an innovative, proprietary technology for the manufacturing of liquid crystal mixtures and special-purpose liquid crystal emulsions. This manufacturing process was developed based on Continuous Liquid Crystal Film (CLCF) technology. The technology used by BRASTER S.A. is protected by international patents.

⚠ Improper operation of the device which is inconsistent with the User Manual may pose risk to the health and life of the user or third persons. You must comply with the User Manual when using the product. Only use the Braster device if you have become familiar with the product and have read the User Manual.

⚠ The device must only be used for the purpose intended by the manufacturer. Arbitrary use of the product which is inconsistent with the User Manual may result in the loss of guarantee rights and claims in case of damage.

1. DESCRIPTION AND INTENDED USE

1.1. DESCRIPTION

The Braster System – in-home breast examination system, hereinafter referred to as the “Braster System”, is a medical device intended for thermographic breast examination. The system consists of:

- the Braster device, which is composed of:
 - the image acquisition device, and
 - three thermographic matrices;
- the Braster Care mobile app;
- the Braster AI automatic interpretation software.

See Fig. 1: Components of the Braster System

1. The Braster Care app
2. The Braster device
3. The Braster AI software

During the examination apply the Braster device to the breast area. The device is fitted with a liquid crystal foil which creates color images when it comes in contact with the surface of the breasts. At the end of the examination the acquired thermographic images are stored in the mobile app and sent to the telemedical center, where they are automatically analyzed by Braster AI. Should any potentially dangerous pathological changes be indicated, you will be instructed to consult your doctor.

The Braster medical device does not perform measurements, but instead records temperature distribution across the breast, based on which the thermographic images are analyzed.

⚠ The Braster examination DOES NOT REPLACE EXISTING DIAGNOSTIC METHODS CURRENTLY USED IN CLINICAL PRACTICE (SUCH AS MAMMOGRAPHY AND ULTRASOUND). It is a useful complementary tool to the aforementioned examinations.

1.2. INTENDED USE

1.2.1. INDICATIONS

The Braster System – in-home breast examination system is intended for domestic breast examination by women of legal age as an adjunct to examinations such as ultrasound and mammography. The device detects thermal irregularities that require confirmation by diagnostic methods or healthcare consultation.

1.2.2. CONTRAINDICATIONS

The Braster System is not intended for women who are undergoing or have undergone breast cancer treatment, i.e. breast-conserving surgery, mastectomy (unilateral or bilateral), systematic treatment (chemotherapy, hormonal therapy, biological therapy) and radiation therapy.

Temporary contraindications against using the Braster System:

- pregnancy or breastfeeding (up to three months after weaning);
- general infection, with a body temperature of or in excess of 38°C;
- breast infection with pain, skin redness and bruises (when the said symptoms are present);
- the second phase of the menstrual cycle – due to changes in breast physiology in the menstrual cycle, it is best to perform the thermographic examination between day three and day twelve of the cycle;
- inadequate temperature (too hot or too cold) in the room where the examination is to be performed – the ambient temperature must be between 20°C and 25°C;
- surgical procedure in the breast area with benign lesion diagnosis:
 - fine-needle biopsy (FNB) – up to four weeks after the procedure,
 - core-needle biopsy (CNB) or Mammotome's breast biopsy – up to 6 months after the procedure,
 - breast tumor resection – up to 12 months after the procedure;
- aesthetic implant placement, filler injections (e.g. hyaluronic acid) and lipotransfer – up to 12 months after the procedure.

2. OPERATING PRINCIPLES OF THE DEVICE

The Braster device uses a type of technology known as contact thermography. Contact thermography is a medical imaging technique for detecting changes in tissue metabolism. In medicine, thermography is based on a phenomenon known as the "dermo-thermal effect", in which the thermal processes that occur in the organ examined (inside the body) manifest on the surface of the skin as temperature anomalies.

The neoplastic process is characterized by a higher metabolic rate among other things through the formation of a dense pathological capillary network. Pathological lesions with suspected malignancy have a characteristic morphological structure and higher temperature than normal areas of the body. As a result, they are shown in the thermogram as areas of different colors.

Contact thermography is a complementary examination. It complements commonly performed examinations such as ultrasound and mammography.

3. PACKAGE CONTENTS

The package contains:

- an image acquisition device, model: BRA-1.0 (see Fig. 2A),
- a set of three thermographic matrices numbered 1, 2 and 3, model: TMS-1.0 (see Fig. 2B),
- a mini USB cable,
- a cleaning cloth,
- a user manual,
- a quick start guide.

4. OVERVIEW OF THE BRASTER DEVICE INTERFACE

See Fig. 3: The Braster device interface

1. Grip
2. Examination indicator
3. Power status
4. Mini-USB slot
5. ON/OFF button

ON/OFF Button – turns the device on/off

- to turn the device on press the button and hold it for 2 seconds.
- after the examination is finished or after 10 minutes of inactivity the device will turn off automatically. To turn the device off in emergency mode press the button and hold it for 5 seconds.

Grip – activates acquisition of thermographic images

- The grip is pressed – acquisition of thermographic images is active (activated Examination indicator).
- The grip is not pressed – acquisition of thermographic images is not active.

Examination Indicator Status	Power Indicator Status	Description
No light	No light	The device is off
No light	Blinking green	The device is on
No light	Steady green	The device is connected with a mobile device
Pulsing white	Steady green	Thermogram acquisition in progress
No light	Steady orange	Charging in progress
No light	Blinking orange	The examination cannot be performed. The battery level is too low.
Steady white	No light/orange	Firmware upgrade mode
Blinking white	No light/orange	Firmware upgrade in progress

Tab. 1. Indicator colors in user interface

Mini-USB slot – is located under the rubber cap. To access the slot, you need to raise the cap. The mini-USB slot is intended for:

- updating software for the image acquisition device,
- battery charging.

Device label – placed in the shadowed chamber of the device, visible when the matrix is removed. The label contains the serial number of the device, which is also the name of the WiFi signal broadcast by the device, and password (see Fig. (4A): Position of the label, and (4B): Sample label with name of the WiFi signal and the password: 1. Name of the WiFi signal broadcast by the device, 2: Password to WiFi signal broadcast by the device).

⚠ Do not remove the label. Write down the data which is on the label and keep it in a safe place.

5. MATRIX REPLACEMENT

Step 1 (Fig. (5A)) Pull the edge of the matrix in the place where there is a notch in the body of the device. Preferably hold the notch on the matrix with your thumb and support the matrix from below with your other fingers.

Step 2 (Fig. (5B)) Pull the matrix off the device.

Step 3 (Fig. (5C)) Attach the matrix, pushing it parallel to the housing. Preferably hold the notch on the matrix with your thumb and support the matrix from below with your other fingers.

6. DEVICE PREPARATION PRIOR TO EXAMINATION

6.1. PREPARATION OF THE BRASTER DEVICE BEFORE THE FIRST EXAMINATION

- After unpacking the device, check it for completeness. The box should contain: the image acquisition device, three matrices (numbered 1, 2 and 3), a mini USB cable, a user manual, quick start guide and a cleaning cloth. If you find that any of those elements are missing, immediately fill in the complaint form available on the Braster website.
- The matrix is a part of the device, which means that it comes into direct contact with the skin. Therefore, before using, clean and disinfect the inner and outer surfaces of all matrices (as per the description and by using the measures specified in chapters 11 and 12 of the User Manual).
- Charge the device for at least 30 minutes before the first examination.**
- Download the free Braster Care mobile app** onto your smartphone or tablet and create your account on myaccount.braster.eu in order to perform an examination. You can do this while the battery is being charged. The instruction on how to download the mobile app include:

- Download the Braster Care app from the App Store, if your mobile device has the iOS operating system (i.e. is an iPhone or an iPad), or from Google Play, if your mobile device has the Android operating system;
- Install the Braster Care app on your mobile device;
- Start the Braster Care app on your mobile device and read the INTRO;
- Click the SIGN UP link at the bottom of the screen.

e. Register your individual account as instructed (please note, that we recommend registering and using your individual account on PC's or laptops, since it has been designed for large screens):

1. If after clicking the SIGN UP link you are not redirected automatically, go to www.braster.eu and click the MY ACCOUNT button;
2. Fill in the data. In the *type of account* field select "Braster – for home use" and click the SIGN UP button;
3. Go to your e-mail account and find the e-mail from us with the activation link. If you cannot find it in the main folder, check the "Spam" folder;
4. After you have activated the account by clicking the activation link, type your login (the e-mail address which you used during registration) and password (the one which you used during registration) on the login page, and click LOG IN;
5. Fill in all medical and personal data in the survey.
If you do not complete this survey the first time you log in, you will not be able to perform the examination;
6. Activate your examination package by going to the tab "Activation codes" and clicking the button "Activate the package";
7. Log out from My Account after completing the registration.

f. When charging is complete, start the app and follow the instructions described in chapter 7.

6.2. PREPARATION OF THE BRASTER DEVICE BEFORE SUBSEQUENT EXAMINATIONS

a. When preparing for an examination, first clean the surface of the liquid crystal matrices. You should clean all of the matrices, with particular attention to the surface on the inner side of each matrix. It is important to use the recommended cleaning and disinfection agents and thoroughly clean the surface of the foil so as to not leave any fingerprints, smudges, etc. as it can alter the interpretation of the thermographic examination (for more information see chapter 11 titled "Disinfection" and 12 titled "Cleaning and Maintenance").

b. When the matrices have been cleaned, start the app and follow the instructions provided in chapter 7.

7. EXAMINATION

7.1. IMPORTANT INFORMATION

 **Do not use the device while it is being charged.**

Before the first examination:

- download the free Braster Care app from Google Play or the App Store onto your mobile device;

- make sure that you have bought the examination package – if you have not, you may purchase it in the Braster e-store or, depending on the country, the e-store or other outlets of the local distributor.

REMEMBER

 **Do not perform an examination in a room where the temperature is lower than 20°C or higher than 25°C.**

 **Due to changes in breast physiology during the menstrual cycle, it is best to perform the thermographic examinations between day three and day twelve of the cycle.**

 **If you no longer menstruate, or your menstrual cycles are irregular (fluctuations of more than 10 days), we recommend performing the examination at equal intervals once a month; e.g., between the 1st and 5th day of the month or on the 1st weekend of the month.**

1. On the day before the examination:

- avoid procedures which may increase blood circulation in the breast area (physiotherapy, massages, etc.).

2. On the day of the examination:

- avoid procedures which may increase blood circulation in the breast area (physiotherapy, massages, etc.);
- make sure that you are between days three and twelve of your menstrual cycle or, if you do not menstruate, that about 30 days have passed from your previous exam;
- do not take a bath or a shower within approximately two hours before the examination;
- do not consume any alcoholic beverages;
- do not use peelings on your breasts and/or neckline and do not perform armpit depilation, as this may irritate your skin;
- refrain from sources of heat or cold, such as heaters or air conditioners, for at least thirty minutes before and during the examination;
- avoid intense physical effort for approximately two hours prior to the examination;
- do not use cosmetics and do not wet the skin on your breasts within two hours prior to the examination – your skin should be clean and dry;
- do not have a sauna or a steam bath.

7.2. HOW TO PERFORM AN EXAMINATION

7.2.1. PREPARATION

- Place the Braster device in front of you. Make sure it is turned off. Have at hand your mobile device with the Braster Care app installed (as stated in chapter 6 of the User Manual). Check that your mobile device is connected to the Internet.

- Start the Braster Care mobile app on your smartphone or tablet.
- Log into the app by entering your login and password (which are the same as for your individual account on myaccount.braster.eu).
- Click START to begin the examination. You do not need to undress at this stage yet.

During the examination the mobile device will display a navigation screen with all of the stages. The stage you are currently going through will be highlighted in the app. Click the name of the stage to begin. The navigation screen will be displayed after each stage. Note that you must go through all of the stages of the examination one by one. The app will not let you skip any stage. Once you have finished a step, you will not be able to return to it.

7.2.2. QUESTIONS

In this part of the examination you will answer questions which will check whether you are eligible to perform the examination on that particular day. These questions will also be useful for interpretation of the results. Some of them will be YES or NO questions. If your answer enables you to perform the examination, you will be redirected to the next screen. However, if you are unable to perform the examination, an explanatory note will be displayed. You can always return to the previous question by clicking BACK.

In some of the questions you will select your answer from a number of options. Always remember to answer truthfully, as your answers affect the reliability of the results.

7.2.3. TURNING THE DEVICE ON/OFF

Turn the device on by pressing the ON/OFF button for two seconds. When the device is on, the battery icon will flash green. When the device is connected to your mobile device, the battery icon will be steady green.

In normal conditions the device turns itself off automatically once the examination is completed. To switch the device off in the emergency mode, press the ON/OFF button and hold it for five seconds.

The device turns off automatically after ten minutes of inactivity. Therefore, if you find it necessary to interrupt the examination for more than ten minutes, you will be required to turn the Braster device on and connect it to the mobile device once again to continue.

7.2.4. DEVICE CONNECTION

Establishing a connection between the Braster device and a mobile device differs slightly for Android and iOS. To connect the Braster device follow the instructions displayed on subsequent screens on your mobile device.

Position the device to see the label with the name of the Braster WiFi signal and password. The label is inside the device and is visible only when the matrix has been removed (Fig. 4A & 4B).

7.2.4.1. CONNECTING TO AN IOS DEVICE

Click "Settings" on the connection screen. Go to the WiFi settings. From the list of WiFi signals select the Braster signal specified on the label and enter the password. Return to the app. A message will display information whether a connection has been established successfully.

Do not worry if you went back to the main screen of your mobile device and not to the app directly, you may go back to the application again. At this point you should already be connected to the device.

If, for some reason, you are unable to go to the WiFi settings on the connection screen in the app, leave the app, go to "Settings" in your mobile device, select the "WiFi" tab and select the signal of your Braster device and enter the password. Afterwards, you may return to the app.

7.2.4.2. CONNECTING TO AN ANDROID DEVICE

Select the signal of your Braster device on the connection screen and enter the password (the signal's name and password are given on the device label located inside the device). A message will display information whether a connection has been established successfully.

The app will remember your login and password, so there will be no need for you to re-enter them when performing an examination in the future.

YOUR MOBILE DEVICE WILL NOT HAVE ACCESS TO THE INTERNET WHEN IT IS CONNECTED TO THE BRASTER DEVICE

7.2.5. ACCLIMATIZATION

The next stage of the examination is a six-minute acclimatization, i.e. regulation and stabilization of breast skin temperature. To do this, you will be required to undress from the waist up. Acclimatization is key if you want to obtain good quality thermographic images during the examination. Only with a properly conducted acclimatization can you be sure that the results will be reliable.

To complete the acclimatization process, follow the instructions displayed on subsequent app screens.

7.2.6. MATRIX SELECTION

At this stage, you need to select the matrix for the examination. Your Braster device was provided with a set of three matrices. Each of them works in a different range of temperatures. Remember that your body temperature depends on, among other factors, the day of your menstrual cycle, your age and metabolism. In order to produce a high-resolution thermographic image that will show the distribution of temperatures across your breasts, you will have to select a matrix before each examination.

To select a matrix, make one application of the Braster device to the central area of each breast. Make sure, that the whole matrix adheres tightly to the body. The app will guide you step by step through this process.

First place matrix No. 2 on the device (the number of the matrix is stated on the plastic frame). Follow the instructions displayed on subsequent screens.

! START THE MATRIX SELECTION WITH YOUR RIGHT BREAST.

When instructed apply the device centrally to your breast and press the grip (see Fig. 3) to see the location of the grip). The clicking sound means that the grip has been pressed correctly.

Remember to hold the grip down as long as the examination is in progress. Do not move the device across your breast, hold it firmly in one place. When instructed, release the button and move the device away from your breast (see Fig. 6) device application to the breast).

! REPEAT THESE STEPS FOR YOUR LEFT BREAST.

If matrix No. 2 is suitable for you, you can start the examination. If it does not suit you, you will be asked to remove matrix No. 2 and repeat the applications with matrix No. 1 or 3. You may be asked to return to matrix no. 2 if it is more suitable for the patient's body temperature. Once the appropriate matrix has been selected, you can start the examination.

7.2.7. EXAMINATION

Before you start the examination, read the instructions displayed when you select "Examination" on the navigation screen. From those instructions, you will learn exactly what to do during the examination and how to apply the device to your breasts.

To ensure top quality thermographic images during the examination:

- apply the device perpendicular to your breast (see Fig. 7) correctly applied device);
- do not lift your breast;
- do not move the device across your breast, instead hold it firmly in one place (see Fig. 8) improperly applied device);
- do not move the device away until instructed;
- keep the grip pressed as long as the examination is in progress;
- do not touch your breast with your hands during the examination.

Depending on the breast size which you entered in "My data" tab in your individual account on myaccount.braster.eu, you will be required to make three or five applications to each breast.

ALWAYS BEGIN WITH YOUR RIGHT BREAST

This is very important, as the thermographic images of your breasts produced in individual examinations are compared to one another. We compare the image(s) from current examination with images produced during your last five examinations. By comparing images produced during the past examinations we can observe if there have been any changes in structural or thermal asymmetry over time.

Breast size 1, 2 or 3

If your breast size is 1, 2 or 3, see Fig. 9A) for size 1 or 2 and Fig. 9B) for size 3. During the examination you will have to perform:

- 1) One central application to the breast, making sure that the lower edge of the matrix, i.e. the black part (the thermographic foil), adheres to your breast (not your abdomen). Do not push your breast up.

2) One upper application to the outer part of your breast, on the border with your axilla, making sure that the edge of the matrix, i.e. the black part (the thermographic foil), touches upon the lateral edge of your breast and does not extend beyond it.

3) One upper application to the inner part of your breast, making sure that the edge of the matrix, i.e. the black part (the thermographic foil), does not extend to your other breast and your sternum.

The applications may overlap, but do not worry if in your case they do not overlap as shown in the illustrations. The crucial thing is to make sure that your entire breast is covered.

Follow the instructions displayed on the screen of your mobile device. Each application takes about 15 seconds which may be extended for transfer if the WiFi conditions are poor (a lot of other transmitters or radio noise). The time interval between applications lasts approximately 10-15 seconds (depending on your breast temperature and the ambient temperature). The matrix should not be in contact with your body during this time.

Breast size 4 or 5

If your breast size is 4 or 5 see **Fig. 10A** for size 4 and **Fig. 10B** for size 5, during the examination you will have to perform:

1) One central application to your breast, making sure that the lower edge of the matrix, i.e. the black part (the thermographic foil), adheres to your breast (not your abdomen). Do not push your breast up.

Divide the breast into four equal quadrants, as if there was a vertical and a horizontal line crossing the center of your breast.

Now perform:

2) One application to the lower-outer quadrant of your breast, making sure that the lower edge of the matrix, i.e. the black part (the thermographic foil), adheres to your breast (not your abdomen). Do not push your breast up.

3) One application to the upper-outer quadrant of your breast, on the border with the axilla, making sure that the edge of the matrix, i.e. the black part (the thermographic foil), touches upon the lateral edge of the breast and does not extend beyond it.

4) One application to the upper-inner quadrant of your breast, making sure that the edge of the matrix, i.e. the black part (the thermographic foil), does not extend to your other breast but runs across the centerline of your sternum.

5) One application to the lower-inner quadrant of your breast. The lower edge of the matrix, i.e. the black part (the thermographic foil), should adhere to your breast (not your abdomen). Do not push the breast up.

The applications may overlap, but do not worry if in your case they do not overlap as shown in the illustrations. The crucial thing is to make sure that your entire breast is covered.

Follow the instructions displayed on the screen of your mobile device. Each application takes about 15 seconds which may be extended for transfer if the WiFi conditions are poor (a lot of other transmitters or radio noise). The time interval between applications lasts approximately 10-15 seconds (depending on your breast temperature and the ambient temperature). The matrix should not be in contact with your body during this time.

7.2.8. SENDING THE RESULTS TO THE TELEMEDICAL CENTER

Once you have made all applications, you will be informed that the examination has come to an end and that you are able to send your collected data to the manufacturer's telemedical center for analysis. Please note that before you send the data, you should first disconnect your mobile device from the Braster device and connect to a WiFi network. Due to the size of the thermographic images produced during the examination, it is not recommended to send them via mobile data but by a WiFi network. The examination results will be sent to your account on myaccount.braster.eu within two business days.

If you select "Finish" after having sent the examination, you will be redirected to the start screen, where you can check the status of the examination ("Sent for analysis"). When the results of the examination have been sent to your account from the telemedical center, the status will change to "Result available".

You do not need to send the collected data for analysis immediately after completing the examination. You can send them whenever it is convenient for you, when you have access to Internet (preferably via WiFi). When you are ready, go to the app and select "List of examinations", click on the last exam and choose whether to send it or delete it. Please note that you must either delete the results or send them for analysis before you carry out a new examination, otherwise the system will not let you perform your next exam.

The app will let you know when the result of your examination is available. To read it, go to your individual account on myaccount.braster.eu.

8. TROUBLESHOOTING THE MOBILE APP

In certain circumstances, carrying out an examination will not be possible. Below you may find tips on how to troubleshoot certain problems.

8.1. The Braster device uses WiFi technology to transmit data to the mobile device. WiFi technology, by design, relies on sharing radio bandwidth with other users. If the WiFi environment in your location is overcrowded and/or many other WiFi

signals are available, performance of the Braster device may be decreased resulting in a longer registration time during applications or even, in extreme WiFi conditions, inability to register a full set of images. In this situation, please go to a place where the WiFi environment is not overcrowded or return to the examination later that day.

- 8.2.** If you have not bought an examination package or have used up all purchased examinations, the app will display an appropriate message and the examination will be impossible. You will have to buy an examination package to continue. To do this, go to the Braster e-store or depending on the country, the e-store or other outlet of the local distributor. If you do not want to purchase an examination package right now, select “Cancel the Examination”.
- 8.3.** If the battery level of your smartphone or tablet is too low, the examination will be impossible to perform until the device has been charged to a level indicated in the message. Select “Cancel the Examination” to return to the start screen.
- 8.4.** If there is not enough disc space in your mobile device to save a complete examination, the examination will be impossible to perform until the appropriate amount of disc space is made available. Select “Cancel the Examination” to return to the start screen.
- 8.5.** If after entering your login and password to the Braster Care app a “Server is unavailable” message is displayed, you should install an additional certificate. Please contact our contact center by e-mail to find out more: contactcenter@braster.eu.

9. CHARGING

Charge the device with a USB-A charger consistent with IEC 60950-1 or IEC 60601-1, charger parameters: DC 5V 1A, USB slot (specification of the feeding device). Switch the device off before charging. For safety purposes, while charging, place the device in a position where you may easily remove the USB cable from the device.

 **The device must not be charging while an examination is in progress.**

 **Do not turn the device on while it is being charged.**

To charge the battery, pull the rubber cap with the USB sign situated on the side of the Braster device next to the ON/OFF button. Connect one end of the USB cable to the Braster device and the other end to the charger (the plug is not included in the set). Connect the charger to a power source (see Fig. 11). Location of the USB slot: **1.** mini-USB slot, **2.** USB slot).

When the device is connected to the charger, the Power status indicator will be lit as steady orange. The indicator turns itself off when the battery is fully charged. Full charging takes approximately three hours. The device must be disconnected from the power source via the USB cable, which must be safely disconnected from the mains supply when charging is complete.

10. SAFE OPERATION

Follow the recommendations below to ensure the safe operation of the Braster device:

- Before you use the device for the first time, charge the battery using the USB cable provided.
- Only use accessories and materials supplied or recommended by the manufacturer.
- Never let the Braster device or the matrices come into contact with water.
- After the examination, store the device with a matrix on. Place the device with the matrix facing downwards, making sure that dust does not get into the matrix. Keep the device in the original box.
- Do not place any items into the image acquisition device, this may cause damage to the internal framework of the device. For any damage caused by the user, the manufacturer will not be held responsible.
- Do not allow the foil on the matrix to come into contact with any sharp objects.
- During the examination, do not place any objects between the image acquisition device and the mobile device. If objects are placed between, the connection between the image acquisition device and the mobile device may be lost.
- The device can only be repaired by the manufacturer’s servicing team. Any other attempt to repair the device will void the warranty.
- Keep the device away from children.

11. DISINFECTION

Before each examination disinfect the inner and outer surfaces of all of the three matrices. Disinfection consists of spraying Softasept or disinfectants based on isopropyl alcohol over entire matrix, on both sides, and cleaning it thoroughly with a disposable cotton gauze or tissue. By disinfecting you make sure that the surface is free of pathogenic microorganisms such as bacteria, viruses and fungi.

 **The device must not be washed or immersed in water. Do not clean the applied parts of the matrices, which come into contact with your breast skin, with any mechanical agents that may cause damage to such surfaces (e.g. brushes) and do not use agents that contain organic solvents (e.g. gasoline, acetone) as, once damaged, matrices will not be fit for further use.**

12. CLEANING AND MAINTENANCE

Wipe thoroughly both surfaces of all of the three matrices with the cloth provided. It is important that the surface of the matrix is clean, i.e. free of any visible contamination (e.g. fingerprints, dust). The inner side of the matrix is technologically matted. The foil surface on the inner side of the matrix bears directional scratches, which have been made deliberately in a technological procedure performed in order to achieve appropriate light diffusion in the working space of the image acquisition device.

 **The image acquisition device and the matrices must be kept in the original packaging, at temperatures between -5°C and +60°C. Do not expose the device to UV radiation (e.g. from UV lamps used for air disinfection or from the sun).**

13. SERVICING AND TECHNICAL SUPPORT

If you require technical support or if you want to report a malfunction of the device or any other unexpected circumstances, contact your local distributor of the Braster System or the manufacturer. Contact details are available on www.braster.eu.

USER RESPONSIBILITIES

The Braster device must be used in accordance with the recommendations given in this User Manual and labels. Do not use the device if it has been damaged. Any parts that are missing, incomplete, damaged or worn out must be immediately replaced at an authorized service point (details are available on www.braster.eu). Any actions connected with repairs or replacements may only be performed by personnel of an authorized service point. The manufacturer is not liable for any damages caused by non-compliance with the user manual.

Warranty

The Braster device is covered by the manufacturer's warranty for 24 months. The warranty shall only be valid if accessories and spare parts approved by BRASTER S.A. are used and the device is used as described in the user manual and according to the intended use.

Any repairs of the Braster device must be made by personnel of an authorized service point. Any repairs performed by unauthorized persons will result in voiding of the warranty.

14. DECLARATION OF CONFORMITY

BRASTER S.A. hereby declares that the Braster System conforms with the essential requirements and other relevant provisions of the Directive 93/42/EEC and the Directive 2011/65/EU. If you would like to receive the declaration of conformity, contact the manufacturer (contact details are given on the last page of this User Manual).

ELECTROMAGNETIC COMPATIBILITY

The Braster device is intended for use in an electromagnetic environment with controlled RF interference. The customer or the user of the image acquisition device may help prevent electromagnetic interference by maintaining the minimum required distance between the mobile device and cellular radio devices (transmitters). This equipment is not subject to the protection from harmful interference and may not cause interference with duly authorized systems.

15. TECHNICAL AND OPERATIONAL SPECIFICATION OF THE DEVICE

Weight of the image acquisition device (excluding matrix)	305 g
Weight of one matrix	87 g
Device size (excluding matrix)	diameter 17.2 cm height 13 cm
Limits of storage conditions	-25°C to +5°C (-13°F to +41°F), and +5°C to +35°C (+41°F to +95°F) in ambient humidity of up to 90%, without condensation, and +35°C to +70°C (+95°F to +158°F) with vapor pressure of up to 50 hPa
Recommended storage conditions	-5°C to +60°C (+23°F to +140°F) humidity between 10% and 90% (without condensation)

Limits of operating conditions	+5°C to +40°C (+41°F to +104°F) humidity between 15% and 90% (without condensation)
Recommended operating conditions	+20°C to +25°C (+68°F to +77°F) humidity between 30% and 85% protected against UV radiation
The limits of the operating conditions guarantee the electrical safety of the device. To protect the thermographic matrices against excess wear and damage, keep them in the recommended operating and storage conditions.	
Maximum approved operation altitude	3500 m above sea level
Approved atmospheric pressure (for the device to be used)	700–1060 hPa
The device can be paired with	A smartphone or tablet with a WiFi module and a RAM of at least 100 MB. For system requirements, go to: https://www.braster.eu/system
Charger	DC 5V 1A with a USB slot and meeting the IEC 60950-1 or IEC 60601-1 standards

16. ADVERSE EVENT AND MEDICAL INCIDENTS

Any adverse event or serious incidents that occur in relation to the device should be reported to the manufacturer on the address given in Section “Contact the Manufacturer” and to the competent authority of the country in which the user is established.

17. FCC REGULATORY STATEMENT

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The Braster device has been tested and found to comply with the limits for a Class B digital device. These limits are designed to provide reasonable protection against harmful interference in a residential installation. The Braster device generates, uses, and may radiate radio frequency energy and, if not installed and used in accordance with the User Manual, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device to an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

⚠ Any changes or modifications not approved by the party responsible for compliance will void the user's authority to operate this equipment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter. The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this device.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment in accordance with FCC rule part 2.1093 and KDB 447498.

18. CANADIAN REGULATORY STATEMENT

The Braster device complies with ISED Canada License-exempt RSSs. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable conditions. The Braster device contains an internal, integrated antenna and it cannot operate in conjunction with any other antenna or transmitter.

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19. DISPOSAL

This symbol is used by the Company as part of the resource-efficient initiative and health and environment protection, in accordance with governmental instructions. The symbol means that the electronic device (including batteries) must not be thrown away together with conventional household waste. Should you need more information on the dedicated waste collection points, please contact your local authorities.

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